Gebrauchsinformation	7
Instructions for use	13
Mode d'emploi	19
Istruzioni per l'uso	25
Instrucción de uso	31
Instruções de uso	37
Gebruiksinformatie	43
Brugsanvisning	49
Användarinstruktioner	55
Informacje o zastosowaniu	61
Информация по применению	67

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference in which case the user will be required to correct the interference at his own expense.

**Caution:** Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**Service Address USA:**

DMG America  
242, South Dean St / Englewood, NJ 07631 / USA

Federal Law (USA) restricts this device to sale by or on order of a dentist or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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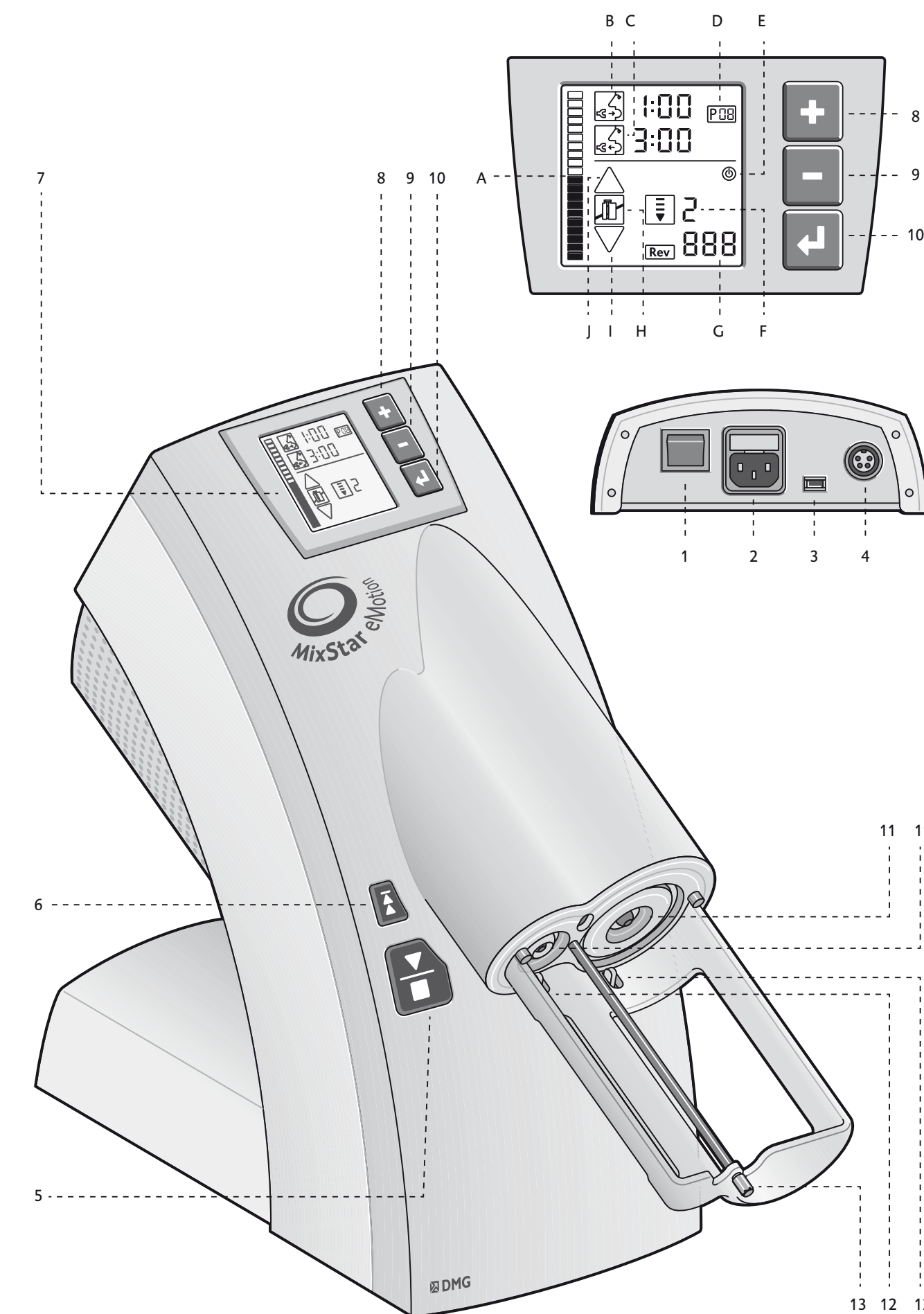
## Table of contents

General information	13
Device and display functions	14
Getting started	14
Preparing a cartridge for use	14
Loading a tray	15
Filling an elastomer syringe	15
Exchanging a cartridge	15
Attaching a mixing tip	15
Setting the timer	15
Program presets	16
Selecting programs	16
Operation without the timers	16
Restart	16
Reset to factory settings	16
Maintenance and care	16
Troubleshooting	17
Packaging	17

## What does the new MixStar offer?

The new DMG MixStar-eMotion is a fully automatic mixing device for impression materials. To accommodate individual preferences the device can be operated at 3 different speed levels. All speed levels are electronically controlled and regulated to ensure a consistent speed even with viscous consistencies. To ease the practice routines for dentists and staff two timers can optionally be programmed for working time and time in mouth settings. And the recommended time values for each process are specified on the pre-filled MixStar cartridges. The mixing process is started by simply pushing the Start button allowing you to guide the tray with both hands. For a hands-free operation an optional foot switch pedal can be connected to the unit. The integrated USB interface is used to upgrade the software.

## Safety instructions

- Please read these instructions for use carefully.
- Please check before operation that the operating voltage indicated on the rating plate is identical with the voltage available.
- Only use properly grounded sockets to connect the device. If you intend to use an extension cable, make sure the ground line is uninterrupted.
- Always use MixStar on a flat surface.
- If the unit has been stored below + 10 °C/50 °F please do not operate the MixStar-eMotion has reached room temperature. Otherwise water from condensation build-up may lead to a short circuit of the unit.
- Please use only MixStar compatible products with the MixStar-eMotion. We cannot guarantee proper function with other products.
- Use the MixStar-eMotion only for its intended purpose.
- Use only power cords that are approved for use in your country.

## Storage and transport

Store and transport the unit between -20 °C to +60 °C (-4 °F to +140 °F) and a maximum relative humidity of 80 %. For transporting the unit we recommend using the original packaging.

## Technical data

Housing made from high impact resistant resins, hazard class I, type B	
Dimensions H x W x D	370 x 180 x 430 mm (14.5 x 7.08 x 17")
Weight	8,9 kg (19.58 lbs)
Ambient temperature	+5 to +40°C
Recommended operating temperature	+18 to +40°C
Time for plungers to return to cartridge exchange position from most forward position	11 seconds in cartridge exchange position
Power supply	115 V 30 V
Frequency	60 Hz 0 Hz
Fuse [2 x type]	T 4A, 250 V 2A, 250 V
Output	200 W = 200 VA
Duty cycle	up to 2 min.
Pause	minimum 5 min.
USB connector	Mini USB B type 5 pin
Foot pedal connector	4 pin type MAS 4100, threaded
Foot pedal rated voltage	24V
Foot pedal rated current	0.1A

## Device and display functions

[1]	Power switch
[2]	Power port with fuses
[3]	USB interface
[4]	Foot pedal connector
[5]	Start/Stop button
[6]	Reverse button
[7]	Display
[8]	Plus button
[9]	Minus button
[10]	Menu button
[11]	Plunger disks
[12]	Cartridge recognition
[13]	Drive rod

### Cartridge gauge [A]

Indicates amount of material left in cartridge. The cartridge gauge blinks as long as the level cannot be determined or if the last two level indicators have been reached and only enough material for loading one tray remains in the cartridge.

### Timer 1 [B]

Timer 1 is used for setting the working time for the material used. The timer counts down with the start of the mixing process. A long acoustic signal indicates the end of the working time.

### Timer 2 [C]

Timer 2 is used for programming the material's setting time. The count-down starts as soon as the mixing process is completed. Three acoustic signals indicate the end of the setting time.

### Program display [D]

The unit provides storage space for 99 timer settings and an easy recall function.

### Standby indicator [E]

If the unit is not used for 10 minutes the display switches to standby mode. By pressing any of the buttons it is reactivated automatically.

### Mixing speed [F]

Unless a menu is selected the mixing speed can be increased by pressing the Plus button [8] or decreased by pressing the Minus [9] button. Three speed levels are available.

Level 1	Slow mixing speed
Level 2	Medium mixing speed
Level 3	High mixing speed

### Software version [G]

The software version currently installed in the unit is indicated in the lower right corner for 2 seconds when the unit is turned on.

### Cartridge indicator [H]

The cartridge indicator shows that a cartridge is inserted and ready for use. If (with the cartridge inserted) the indicator shows a crossed-out cartridge symbol the cartridge is not inserted correctly.

### Forward indicator [I]

If the forward indicator symbol is displayed the plungers can move forward. The symbol blinks when the plungers progress quickly toward the material. During the mixing process the symbol is fully illuminated.

### Reverse indicator [J]

If the reverse indicator symbol is displayed the plungers can move backwards. The symbol blinks when the plungers retract quickly from the material. Once the cartridge exchange position is reached the symbol is no longer displayed.

## Getting started

[Abb. 01]

1. Put MixStar on a flat surface.
2. Connect the power cord to the unit [2] and to the power supply.
3. Switch the unit on with its toggle switch [1] on the back of the device. All indicator lights on the display will blink for approx. 2 seconds following the unit's brief internal self-test. The cartridge symbol is crossed-out if no cartridge is inserted or the cartridge is not seated properly. If a cartridge has already been inserted into the unit the level indicator will blink until the level is determined (after advancing the plungers).

For safety reasons the unit operates only when a cartridge is inserted.

### Using a foot switch (optional)

If the optional foot pedal is used plug the foot pedal connector into the port [4] on the back of the MixStar-eMotion and tighten the screw connection. Pressing down on the foot pedal mixes and delivers the impression material. The process stops when the foot pedal is released.

## Preparing a cartridge for use

[Fig. 02 - 06]

1. Slide locking mechanism into open position. Do not remove the locking mechanism from the cartridge.
2. Remove and discard the cartridge cap.
3. Attach a new mixing tip. Make sure the mixing tip is properly seated.
4. Secure the mixing tip by sliding the locking mechanism to locking position.
5. Leave used tips on the cartridge after use. They serve as a cap.
6. Insert the cartridge from the top into the unit. A short acoustic signal will sound when the cartridge is seated properly.

Fresh cartridges are ready for use after the first 3 cm of impression material have been extruded and discarded.

## Loading a tray

[Fig. 07 - 08]

The simple two-button or foot pedal control leave both hands free to operate the tray.

The mixing speed can be increased or decreased by pressing the Plus button [8] or the Minus button [9], which is possible during extrusion as well.

1. Start mixing by pressing the Start/Stop button [5] or by operating the foot pedal. If Timer 1 is set its countdown will start now. The end of the working time is indicated by a long acoustic signal. If a new Mix-Star cartridge is used discard the first 3 cm of the newly mixed material. Carefully follow the manufacturer's instructions for the selected material. For a clearer display all elements for which no new settings are made (-:-) are hidden after the Start/Stop button [5] is pressed.
2. Tilt the tray slightly under the mixing tip to load it. To avoid entrapment of air leave the mixing tip immersed in the impression material while filling the tray continuously with the material.
3. After the tray has been sufficiently loaded stop dispensing by pressing the Start/Stop button [5] or by releasing the foot pedal. Mixing is stopped at once and pressure on the cartridge is relieved to avoid any further flow of the material. If Timer 2 is set its countdown will start now. 3 acoustic signals indicate when the material can be removed from the mouth. The unit stops dispensing automatically after 2 minutes but can be restarted by pressing the Start/Stop button [5].

### Filling an elastomer syringe

[Fig. 09 - 10]

For filling elastomer syringes we recommend using mixing speed 1. The mixing speed can be increased by pressing the Plus button [8] or decreased by pressing the Minus [9] button. Speed adjustments are possible even during material delivery.

1. Remove the plunger of the elastomer syringe.
2. Attach the elastomer syringe with its rear end to the mixing tip.
3. Start mixing by pressing the Start/Stop button [5] or by operating the foot pedal. If Timer 1 is set its countdown will start now. The end of the working time is indicated by a long acoustic signal.

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4. Stop dispensing into the syringe by pressing the Start/Stop button [5] or releasing the foot pedal. Mixing is stopped at once and pressure on the cartridge is relieved to avoid any further flow of the material.

If Timer 2 is set its countdown will start now. 3 acoustic signals indicate when the material can be removed from the mouth.

### Exchanging a cartridge

[Fig. 11 - 12]

1. The cartridge can only be removed if the plungers are in cartridge exchange position. If necessary, press the Reverse button [6] and wait until the plungers return to their original position. The Reverse indicator [J] blinks during plunger retraction. When the plungers reach the cartridge exchange position the Reverse indicator is no longer displayed.
2. Release the cartridge from the cartridge compartment by pressing with one hand from below the openings and remove it from above. Leave used mixing tip on the cartridge after use. It serves as a cap.
3. Insert the new cartridge into the unit from the top (see »Preparing the cartridge for use«, step 5). In order to use up the material in the cartridge keep a new cartridge with attached mixing tip ready before starting the mixing process. That way you ensure that the remaining material in the old cartridge does not have to be discarded as the working time limit would be exceeded while exchanging the cartridge.

### Attaching a mixing tip

[Fig. 03]

Attach a new mixing tip before each mixing process.

1. Slide locking mechanism into open position.
2. Discard the old mixing tip. If the cartridge was stored for a longer period of time be sure to check the openings for possible blockings and remove them.
3. Align the mixing tip with respective outlets on the cartridge and attach it to the cartridge.
4. Lock the mixing tip with the locking mechanism on the cartridge. Attach a new mixing tip before each mixing process. The device is unable to deliver impression material through material that has cured in the mixing tip. If the mixing tip is not properly attached to the cartridge material may be pressed past the mixing tip and cause soiling.

### Setting the timer

The unit provides memory for 99 timer settings. The timers are used to set the specific working time [B] and time in mouth [C] for the material used. The maximum time setting is 9:59 minutes. An acoustic signal indicates when the set time expires (0:00) and the timer is reset to the previous time setting. The previous time setting kept in memory even when the unit is turned off. You assign your settings to a fixed program memory (D) and access them by recalling the respective program. The unit can be operated with both, one, or none of the timers. The device is supplied with preconfigured programs according to the DMG material specifications.

1. Press the Menu button [10] briefly. The program display is selected and blinks.
2. Select a program memory to store the timer settings. Use the Plus button [8] and Minus button [9] to scroll through the programs. Keeping either of the buttons pressed scrolls through the programs in steps of 10. Pressing one of the two buttons briefly opens one setting at a time.

- Pressing the Menu button [10] again acknowledges the program selection. Timer 1 is selected and blinks. If the timer is selected pressing the Plus button [8] or Minus button [9] will increase or decrease the time setting in increments of seconds to the next minute. Keeping the button pressed counts in increments of minutes. Pressing one of the buttons again sets the seconds. Pressing the Menu button [10] briefly selects Timer 2 which is set as described above. For the working time and time in mouth values please refer to the material manufacturer's instructions. When programming Timer 2 to the specified setting time please remember to add the required transfer time (end of mixing process until insertion into the mouth).
- Pressing the Menu button [10] closes the menu and saves the settings under the respective memory number. A short acoustic signal confirms that the settings are saved.  
If you press the Start/Stop button [5] before saving the settings the menu closes. Unsaved settings will be lost.

## Program presets

Currently the device is supplied with six preconfigured programs.

Program	Material	Working time	Recommended time in mouth
P1	Honigum-Heavy	2:15 min	3:15 min
P2	Honigum-Heavy Fast	1:15 min	2:00 min
P3	Honigum-MixStar Putty	1:45 min	3:15 min
P4	Honigum-Mono	2:00 min	3:15 min
P5	StatusBlue	1:15 min	1:45 min
P6	Silagum-MixStar Putty Soft	2:00 min	3:30 min

These preconfigured settings can be changed. When setting Timer 2 to the specified setting time please remember to add the required transfer time (end of mixing process until insertion into the mouth).

When you reset the device to the factory settings the settings are reverted back to the preconfigured values.

## Selecting programs

- Press the Menu button [10] briefly. The program display is selected and blinks.
- Select the desired program by pressing the Plus button [8] or the Minus button [9], and scroll through the programs. Keeping the button pressed counts in increments of 10. Pressing one of the two buttons briefly opens one setting at a time.
- Pressing and holding the Menu button [10] closes the menu and selects the corresponding program.

## Operation without the timers

If none of the timers are used they can be hidden from the display to provide a better overview. To do that, press the Minus button [9] to set the program display to P— (see »Selecting programs«), and save the settings by pressing the Menu button [10]. After you press the Start/Stop button both timers disappear from the display. This setting will remain active even after the unit is restarted. By pressing the Menu button you can return to the setting mode and reactivate the timer symbols in the display.

## Restart

The unit can be restarted after it was turned off on the main switch and remained off for a minimum of 5 seconds.

## Reset to factory settings

Starting the unit and hold the Plus button [8] and Minus [9] button pressed until the self-test is completed. Previously saved programs will be deleted. The saved programs (see »Program presets«) are loaded. The mixing speed is set to level 2.

## Maintenance and care

Under normal operating conditions the device is maintenance-free. In case of a malfunction, however, please contact your nearest Service Center. With the exception of the plunger disks and fuses there are no user serviceable parts. Spare fuses and plunger disks are supplied with the device.

## Disinfection and cleaning

For your own safety please disconnect the device from the power supply before cleaning and disinfecting the unit.

Use a soft cloth and mild detergent only to clean the device. Do NOT use solvent-based or abrasive cleaning agents as they can damage the plastic components. Cleaning agent must not penetrate into the device.

Please use a normal wipe disinfection to keep the unit and power cord clean. Alcohol-based disinfection agents such as Pursept-A Xpress (Merz) or FD 322 (Dürr) can be used. Pursept-A and FD 322 are not available in all countries.

## Cleaning/replacing the plunger

Should the plunger disks become soiled or damaged they may be cleaned or replaced as follows:

- Retract the plungers to their original position and remove the cartridge.
- Hold the Menu button [10] until you hear an acoustic signal. All symbols except for the cartridge and the forward indicator disappear from the display.  
The device chimes and the Start/Stop button [5] can be operated. Hold the Menu button [10] during the entire process.
- Advance the plungers forward so that plunger discs can be accessed easily.
- Clean: Clean the plunger disks with a soft cloth.

Replace: Unscrew the screws of the plunger disks with the hex wrench provided. Remove the plunger disks from their plungers and replace them.

5. Press and hold the Menu button [10] again, and press the Reverse button [6]. The plungers return to their original position. Plunger disks and hex wrench are included in the accessory »Service Kit«.

### Replacing the fuses

1. Unplug power cable.
2. Take out the fuse drawer [2] above the power port on the back of the device with a screw driver and replace the fuses.

Power input	115 V	230 V
Fuse type	T 4 A, 250 V	T 2 A, 250 V

Replacement fuses are included in the accessory »Service Kit«.

### Troubleshooting

In case of a malfunction the device will display different error codes. E01 indicates a possible operation error. Please proceed as advised on the display. If the malfunction cannot be remedied or if another error code appears please contact us

by phone at +49 40 84 00 60 or by email at [info@dmg-dental.com](mailto:info@dmg-dental.com).

#### Device does not start

- Check if the power cable is connected properly to the device port and the outlet.
- Check the fuses and replace them if necessary. Follow the specifications on the rating plate.

#### Device starts but no impression material is delivered

- Check if a new mixing tip is attached.
- Foil bags: No foil bags in cartridge, foil bag is not opened sufficiently.

#### Impression material flows too slow

- Let impression material warm to room temperature (18 °C/64 °F).
- Foil bags: Check correct placement.

#### Impression material is dispensed, mixer does not turn

- Check if mixing rod is engaged properly into mixing tip.
- Attach a new mixing tip, if hex opening of mixing tip is damaged.

#### Damaged foil bags

- Check plunger disks. Replace damaged plunger disks, tighten plunger disks if loose.

#### Device automatically switches off

- Remove cured material from cartridge openings.
- Check if a new mixing tip is attached.
- Switch off device with main switch for at least 5 seconds.

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